



MammoView™ Sheath/Cannula/Dilator Delivered Sterile- Single Use Device

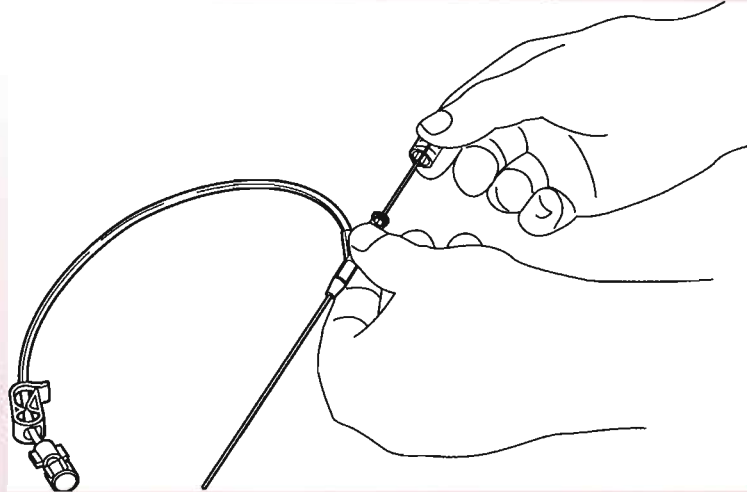

The Sheath Cannula is shipped sterile with the Dilator in place. After insertion into the breast, remove the Dilator from the Sheath by loosening luer fitting. Sheath/Cannula remains in place to receive scope.

MammoView™ Microendoscope

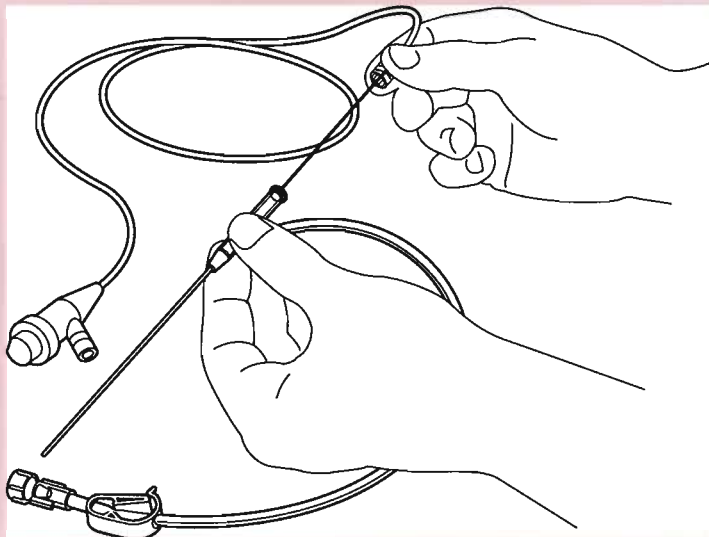

The MammoView Microendoscope is slid into Sheath Cannula after the Sheath Cannula has been placed into the breast and the Dilator has been removed. The scope is secured to the Sheath Cannula by the luer lock fittings. The scope is removed from the Sheath Cannula by loosening luer lock fittings and withdrawing the scope.